

H1108 Facial Steamer

●Product Feature

Innovative Aluminum style appearance with leading structure design,

With stainless steel heating system, built-in UV sterilization tube,

Pull-out water cup for easy operation

Unique vapor arm with aluminum finish, with 180 degree angle adjustment

Sensor touch control, microcomputer control, 3-level vapor adjustment, auto water level detection.

Water level alarm system to avoid heating system or heating tube burning

●Function:



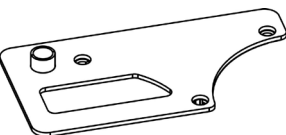
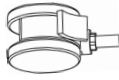
- 1) Soften the dead cells on the surface skin for the next cleaning treatment.
- 2) Stretch skin pore for deep cleaning.
- 3) Permeate into the skin pore, Soften the cumulated oil dirt, blackhead, cosmetic remain and dirt, which make the next cleansing treatment more efficient.
- 4) Help the skin pore to exclude toxin.
- 5) Soften wrinkle temporarily.
- 6) It can accelerate blood circumfluence, add water to cell, and improve the metabolism and self-regeneration of skin cell.
- 7) Applying the proper Chinese herbal medicine during the steam treatment, it can improve the effect of curing acnes, black spot and whiten, nourish skin.




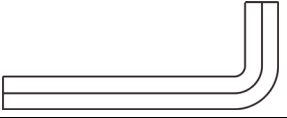




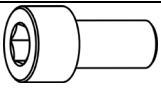




With aromatherapy treatment, stimulating the partial nerve and nursing the physical function in effective way.

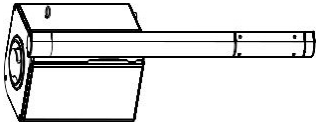●Specifications and Technical Data:

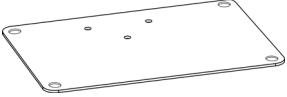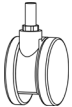Product Name	Micro-Computer Fragrance herbs Facial steamer
Item No.	H1108
Function	Ion spray Facial Steamer
Rated Voltage/Power	AC. 100-120V/750W AC. 220-240V/800W
Working Frequency	50/60HZ
Dimension	89X41X32.5CM
N.W	17KG

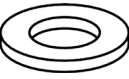
●Packing list:

Item	Product Description	QTY	Item	Product Description	QTY
Host machine		1	Herbal Cover		1
Roof (magnifying glass bracket) components		1	Belt brake casters		2

Roof		1	Without belt brake casters		2
17# open spanner		1	M10 allen wrench		1
φ10 Spring gasket		3	19# open spanner		1
φ10 flat gasket		3	φ12 Spring gasket		4
M10×25 Hex cylinder head machine silk		6	φ12 flat gasket		8
power wire		1 pcs	M12 Cover		4
			Instruction Manual		


5. Standing

●Steamer Structure Description

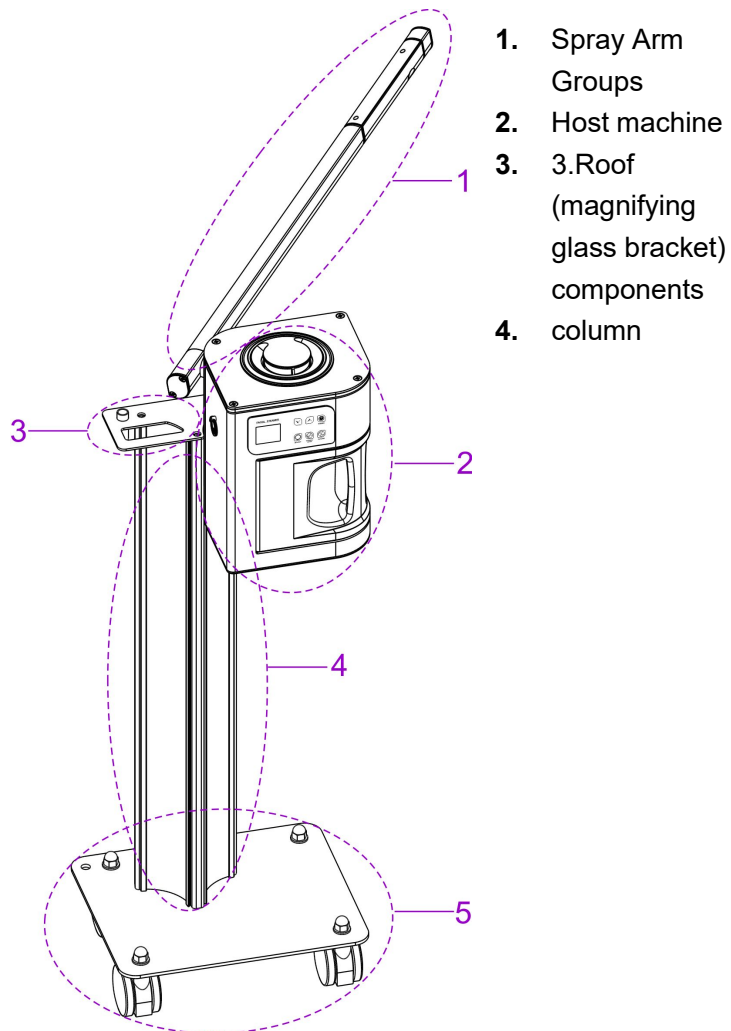


●Spray Arm Groups:

1. Essential oil base
2. Essence oil Cotton
3. Steam Output
4. Spray Arm

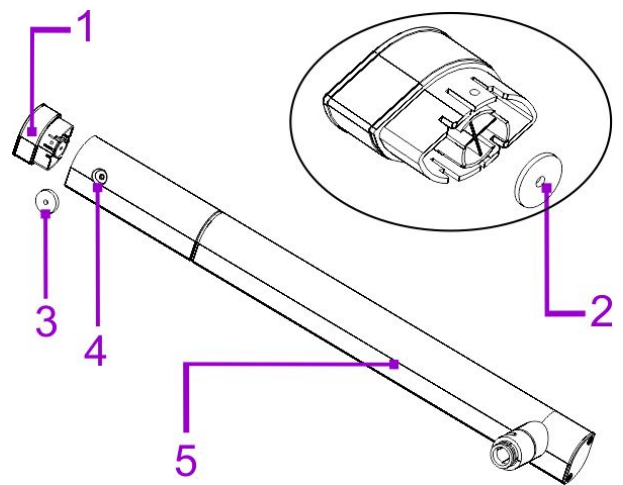


● Host machine Description

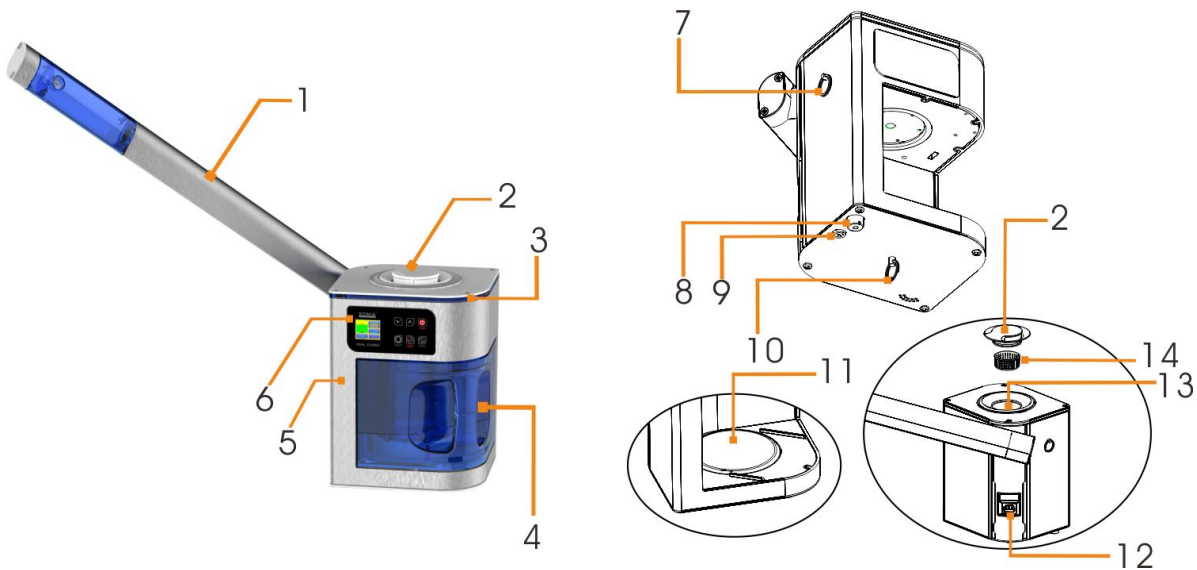


1. Herbal Net
2. Add enough water to cover
3. LED Lamp with
4. Water Cup
5. Host Handle
6. Control Panel
7. Spray arms ring
8. Starter
9. Insurance header
10. Hot plate operative ring
11. Hot plate
12. The power input socket
13. Filler
14. Herbal base

Host machine Description:

1. 1.10×20 Hex cylinder head machine silk
2. Aluminum pillar
3. M12 Cover
4. $\phi 12$ flat gasket
5. $\phi 12$ spring washer
6. Belt brake casters
7. without Belt brake casters
8. $\phi 10$ flat gasket
9. $\phi 10$ spring washer
10. Bottom plate
11. Roof (magnifying glass bracket) components

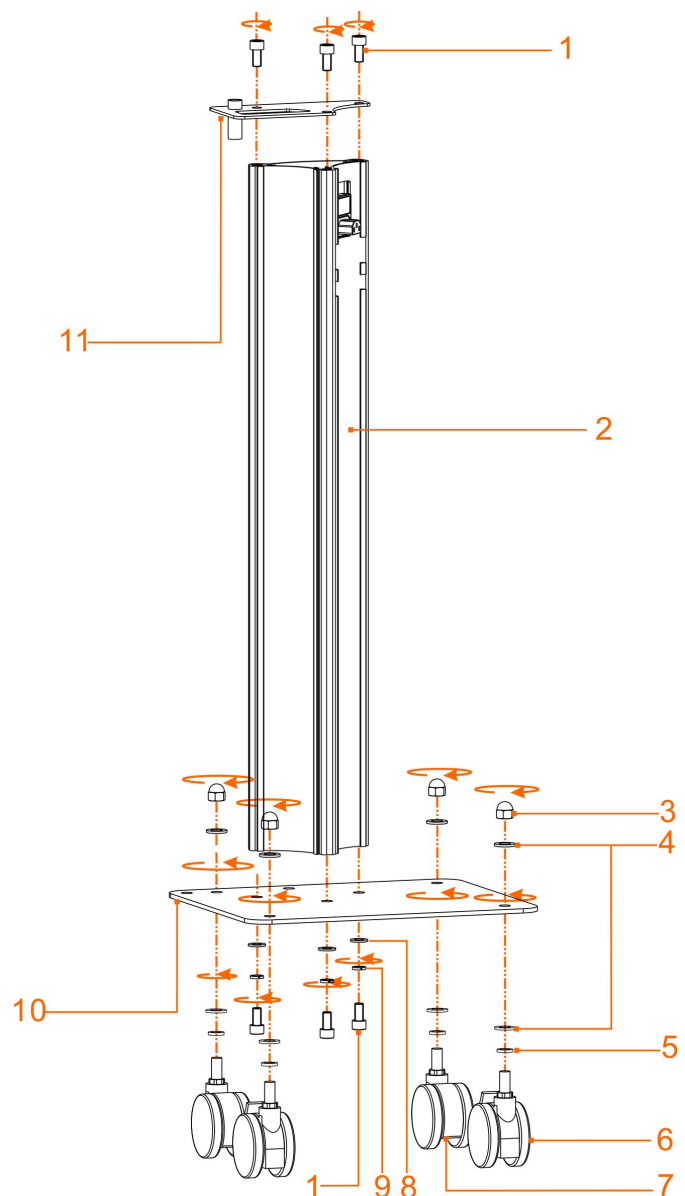

●Assembly Instruction:

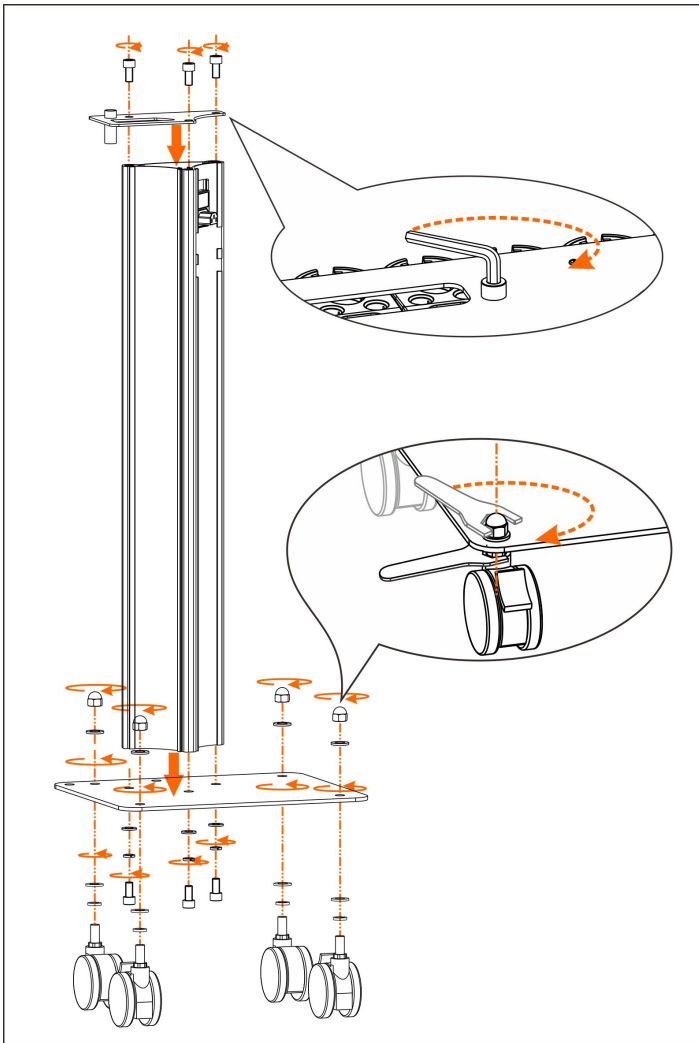

Step 1: install the floor and roof components (attachment)


1.install the casters to the Standing: First, put the casters on $\phi 12$ spring washer, $\phi 12$ flat gasket, then insert into the base plate (pay attention to that the belt brake casters installed in the front and put a $\phi 12$ flat gasket after tighten M12 cover up . While tighten the cover up, please use 17# open-end spanner to fix the nut at the bottom of the floor, then tighten the cover up with 19# open-end spanner.

2.Install aluminum pillar: fit the Aluminum pillar to the three holds on bottom plate, then connect with M10×20 M10×20 Hex cylinder head machine silk, then tighten up screws with M10 allen wrench.


3.Install the roof (magnifying glass) components: fit the Aluminum pillar to the three holes on bottom plate, then connect with M10×20 M10×20 Hex cylinder head machine silk, then tighten up screws with M10 allen wrench.

Step 2: install the sprayer

At first you should put the host machine horizontally to the hole of the column until the column falling down 15 mm naturally.

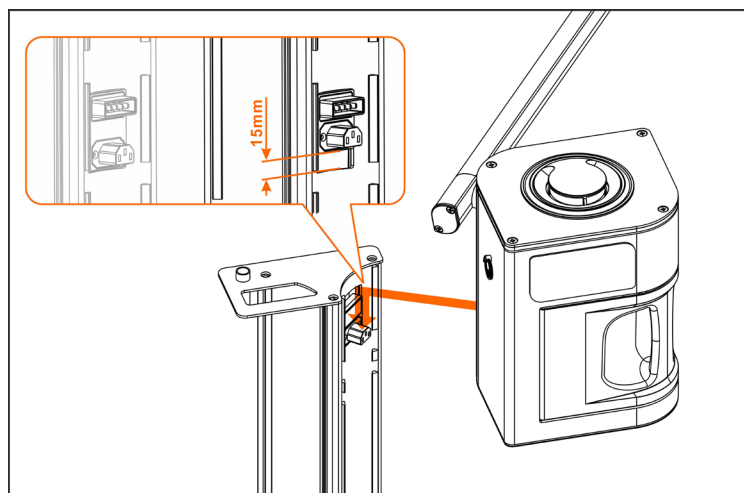


Sprayer panel Instruction:

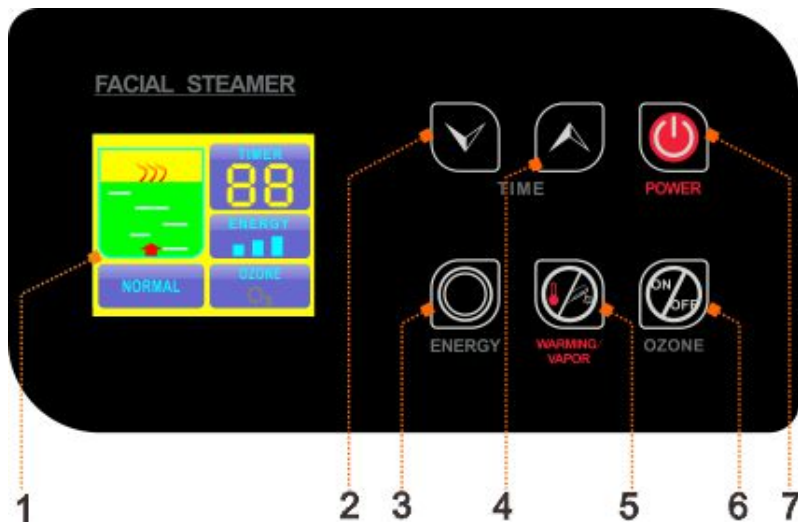

1. working window
2. time decreasing key
3. Adjust Energy Key
4. time increasing key
5. warming vapor start key
6. ozone star key
7. power

Working window Instruction:

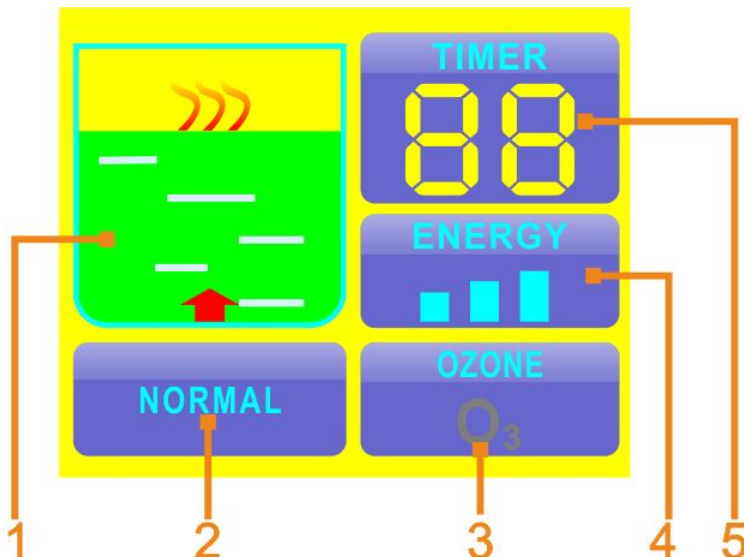


1. water cup working display area
2. warning display area
3. ozone working display area
4. energy display area
5. time display area

●Usage Instruction

1. unlock instruction of Spray arm

When you take out the machine, spray arm is locked. Now you should hold the spray arm and pull out by your hand, rotate the spray arm until to the horizontal position to unlock.


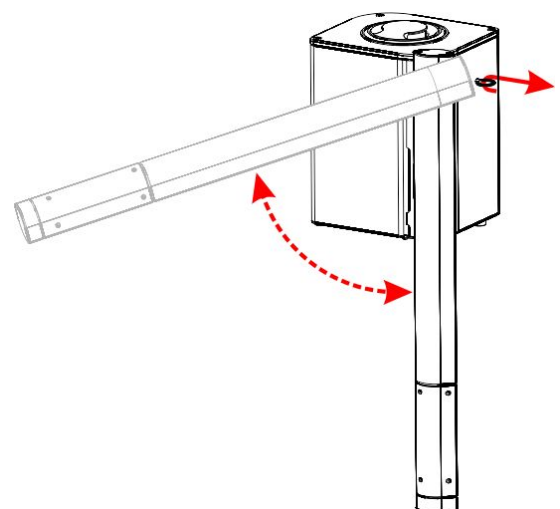


2.Spray arm and spray nozzle adjustment instruction


With unique design spray arm for multi-angle adjustment to adapt different environment.

As the follow picture 1 the Spray arm can adjust right and left 150 degrees.
 As the follow picture 2 the shower nozzle can adjust 360 degrees.
 Pay attention not to adjust too hard.

3.add water to the host machine:

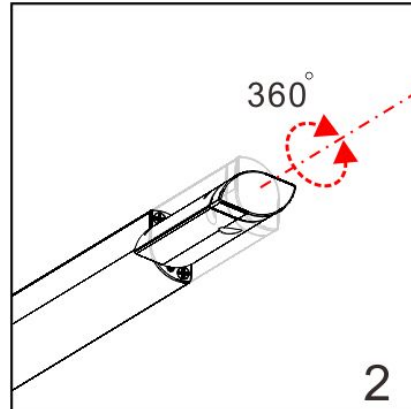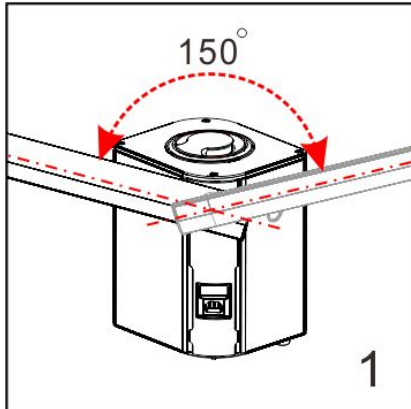


Please lift up the cover, put water to the water cup.
 Water level should not beyond the between the water scale line (pictured at right)

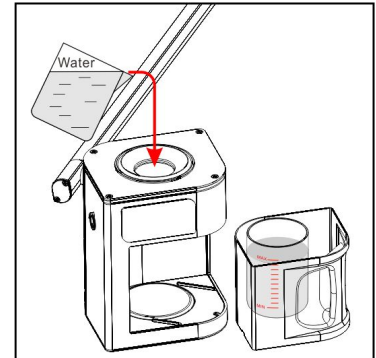

5、 Add perfume essence oil

You should unplug the essential oil base, add perfume essence oil with dropper into the cotton (one to two drops is OK). (Picture at right)


You can change cotton with a tweezer.

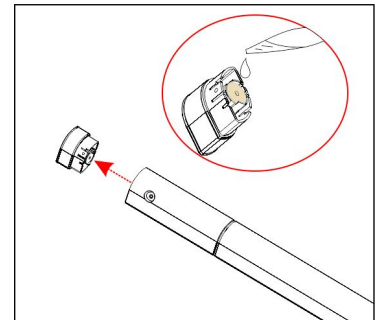

6、 Install and remove water cup:




Please turn down the power of the host machine before removing water cup.
 After the water cooling, Pull hot plate operative ring by one hand, and hold the water cup handle by other hand, then pull out the water cup in horizontal level

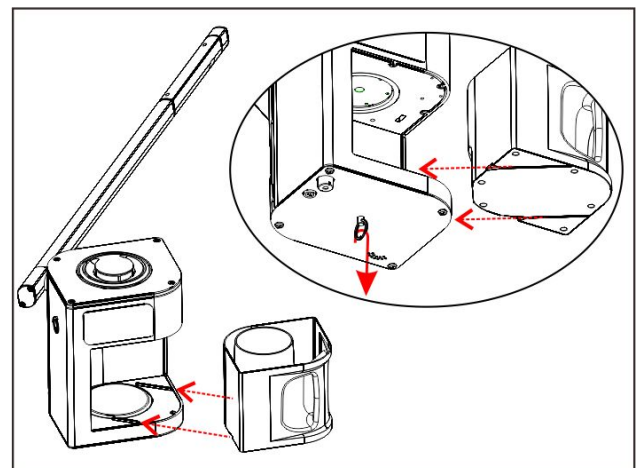Please turn down the power of the host machine before assembling the water cup,
 push down the hot plate operative ring by one hand, and hold the water cup handle by other hand, then push the water cup back to the hot machine.





Control panel operation instruction

1、 Plug the power plug.

2. Press the button  to turn on the host machine, the screen shows the welcome menu, enter the language selection screen, then press  or  button to choose the language.



The host machine will enter the standby mode
Default time display window shows "15" minutes



Energy display window shows  maximum level

"Ozone" display area shows "" symbol dark grey

The symbol of Water cup "" is fixed



3. Hot vapor function:

Press **WARMING/VAPOR**  to turn on the hot vapor function, the display screen shows the hot vapor working status "", the hot machine enters the hot vapor working status; in about 6 minutes, the steamer will come out
Temperature reservation function

Press **WARMING/VAPOR**  to turn on the temperature reservation function, the display screen shows the temperature reservation function working status "", the hot machine enters the temperature reservation function working status


4. Time adjustment


Press  or  to adjust the time.

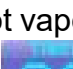
When hot vapor  turn on, the time adjustment range is 1-30 minutes, the default time is 15 minutes ()

When temperature reservation function  turn on, the time adjustment range is 1-90 minutes, the default time is 15 minutes

5. Energy adjustment



choose the energy adjustment button  to adjust the energy. There are three choices for vapor volume, such as high, medium and low

When the energy indication on the energy  is highest

When the energy indication off the energy  is lowest




Please choose the highest energy before the vapor comes out, then adjust the energy to a suitable level.


6、Turn on the Ozone function

Press the  to turn on the Ozone function, the Ozone symbol of ozone indication area  are blue, when the ozone working.

There are ozone smell with the vapor

7、Water level alarm system

The water level detection system work during the hot vapor  or  temperature reservation system on, when the water stay in the normal status ,the host machine heat continuously ;when the water level beyond the lowest water level, the host machine will turn off the heating. And the symbol of the display window  will twinkle all the time

And the buzzer sounds the alarm with “Dee Dee” and lock all the button except the “” button until the water level back to normal status and stop the buzzer and release button to restart the “**WARMING/VAPOR**” function.

●Attention:

1. Before turn on the power of the host machine, please confirm the water cup was screwed down, the water level was between the highest and lowest water level mark, and the herbal medicine cover was covered well in the water hole.
2. If there's needed to add essential oil under working condition, please all it after pause the mist spray.
3. If there's needed to add herbal medicine package during spray, it has to stop mist spray for one minute to open herbal cover. Please open the herbal medicine cover one minute after stop the mist spray.
4. It must be clean the cotton pallet in the essential oil base and the herbal medicine package in the herbal medicine net after stop using aromatherapy or herbal spray everyday.
5. It has to wait the mist came out from the spray hole steady and well-proportionally before facial spray.
6. The spray hole is 40cm away from customer's face and spray steam should be from customer's chin to whole face proportionally to avoid scald the customer's face due to hot spray steam.
7. Ozone has sterilization effect, but it also will harm skin and destroy nutrients of herbal, so it's important to select suitable time for different skin type when using.
8. It's not allowed to take water cup apart when just finish using spray otherwise will cause scald. Please wait water cup cold then take it apart.
9. It's necessary to avoid over adding water, otherwise it's not only make bad effect to the host machine, but also may cause the problem of leakage of electricity
10. If there're problems of water leakage and gas leakage happen underneath the host machine or at the herbal medicine cover, remember cut off the power of host machine first and then check whether the water cup or herbal medicine cover well assembled. If the problems still don't be solved, please contact local agency for further

Solution:

11. It should have good ground connection for the power plug of host machine.
12. When item H1102 under continuous working mode, if water level under lowest mark without adding water, the temperature protection system will react right away to cut off the power temporarily. At this time should add amount of water in time, otherwise temperature protection system would cut power again and

again until burnout the heating system. The host machine will return to normal working condition after adding water for 1-3 minutes.

●Warning!

1. It's forbid to put oily material into water cup or herbal net; it may cause the problem of water spray.
2. It's not allowed spray from the above of the face by the steam hole, otherwise may causing scald.
3. It's not allowed to take the herbal cover and water cup apart under machine is spray.
4. It's not allowed to put this machine in the over humidity environment to work, for example: bath room etc.
5. It's not allowed working under projective sunshine, otherwise will cause many kind of unsteady conditions.
6. If there's any exceptional problem, please contact the agency to let professional maintainer to repair instead of opening the host machine cover to repair by you.
7. If power cord is destroyed, it's better to choose the accessories from the original factory. It's not allowed to repair by un-professional personnel; you should go to appointed repair agency to change.

●Maintenance

- Clean heating groups one time every week. (Tips: Mix some acetic acid or citric acid with water to the water cup for washing, after heating for one minute then using clean water to wash heating tube.)
- Pay attention to the water level of the water cup frequently.
- Cleaning the machine after using, avoid accumulation by beauty leftover.