INFRARED LAMP MANUAL

DDUUEETT EUROPE would like to thank you for choosing our products. Thank you very much for purchasing our infrared lamp. The infrared lamp is ideal for promoting blood circulation, relieving muscle pain, reducing fat, and promoting sweating by eliminating toxins. It improves the body's immune system. Before using the infrared lamp, please read the following instructions.

CONTENTS

- Infrared lamp x 1 unit
- Clamp support x 1 unit
- Instruction manual x 1 unit

CORRECT USE OF THE DEVICE

- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation of DDUUEETT EUROPE SL.
- DDUUEETT EUROPE SL shall not be held liable for any material or personal damage due to the misuse of the product.

HOW TO USE

- **Clamp version:** Attach the clamp to the edge of the horizontal surface of the table or any other horizontal surface. Insert the lower end of the arm into the hole in the clamp. Plug in the infrared lamp.
- **Floor version:** Place the floor stand on the horizontal floor. Insert the lower end of the arm into the hole at the top of the floor stand. Plug in the infrared lamp.

IMPORTANT INFORMATION FOR A SAFE USE

- Indoor use only.
- Environment temperature -5°C~40°C.
- Humidity: ≤95%.
- Never disassemble the infrared lamp unless you are a qualified electrician. If the external flexible cable or wire of the lamp is damaged, it should be exclusively replaced by the manufacturer, his service agent, or a similar qualified person, to avoid danger.
- The plug must be removed while the infrared lamp is being repaired.
- Position the power cable properly so that no one can trip over it and be sure to plug it into a grounded plug.
- The device and accessories should be kept out of the reach of children.

CLEANING AND MAINTENANCE

- Before cleaning the device, unplug it from the power supply.
- Never touch the device or the network cable with wet hands.
- For maintenance or repair of any part of the product, please contact your distributor or specialised technician.

RECYCLING

If you need to dispose of the product or any of its parts or accessories, please check and respect the laws and regulations applicable to waste management and environmental protection.

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